

# DTX Studio™ Clinic 3.2 Shortcuts — MacOS®



## DTX Studio Home

Esc	F1 Help	F2	F3	F4	F5	F6	F7	F8	F9	F10	F11	F12	Print Screen	Scroll Lock	Pause Break	<ul style="list-style-type: none"> <li>■ Patient List</li> <li>■ Image Preview</li> </ul>					
	1	2	3	4	5	6	7	8	9	0	← Backspace		Insert	Home	Page Up	Num Lock	/	*	-	Zoom out	
Tab	Q	W	E	R	T	Y	U	I	O	P			Del	End	Page Down	7	8	9	+	Zoom in	
Caps Lock	A	S	D	F FMX	G	H	J	K	L Level Window	Enter						4	5	6			
↑ Shift	Z Zoom in	X Take IO X-ray	C Capture	V	B Bite-wings	N	M	<	>	Export images	Shift						1	2	3	Enter	
fn	Ctrl	Option	Cmd								Cmd	Option					0	.			
													← Prev. image	↑	↓	→ Next image					



## DTX Studio Clinic

Esc	F1	F2	F3	F4	F5	F6	F7	F8	F9	F10	F11	F12	Print Screen	Scroll Lock	Pause Break	<ul style="list-style-type: none"> <li>■ 2D</li> <li>■ 3D</li> </ul>					
	1	2	3	4	5	6	7	8	9	0	← Backspace		Insert	Home First slide	Page Up 10 up	Num Lock	/	*	-	Zoom out	
Tab	Q Close viewer	W	E Crisp filter	R	T Text	Y	U	I	O	P			Del	End Last slide	Page Down 10 down	7	8	9	+	Zoom in	
Caps Lock	A Edit axis	S Spot-light	D	F FMX Point Focus	G	H	J	K	L Level Window	Enter						4	5	6			
↑ Shift	Z Zoom in	X Take IO X-ray	C Capture	V	B Bite-wings	N	M Measure	<	>	Import images	Shift						1	2	3	Enter	
fn	Ctrl	Option	Cmd	SmartFocus SmartFocus							Cmd	Option					0	.			
													← Prev. image	↑ 1 slide up	↓ 1 slide down	→ Next image					

Adjust brightness and contrast

Rotate 3D view

## Extended Shortcuts

### DTX Studio Clinic

Shortcut	Description
----------	-------------

#### General Shortcuts

Esc	– Restore down maximized window – Finish Spotlight action – Close dialog or wizard
-----	--

Cmd + Q	Exit application
---------	------------------

Cmd + Y	Redo
---------	------

Cmd + Z	Undo
---------	------

Cmd + S	Save diagnosis
---------	----------------

#### Capture

Enter	Capture with intraoral camera (when the livestream is active)
-------	---

#### Viewer Options

Cmd + I	Show / Hide viewer information
---------	--------------------------------

Cmd + M	Maximize / Restore down
---------	-------------------------

Cmd + P	Print viewer images and content
---------	---------------------------------

#### Object Interactions

Backspace, or Del	Delete
-------------------	--------

↑ ↓ → ←	Translate the selected implant in the plane of the reslice
---------	--

Cmd + ↑ ↓ → ←	Translate the selected implant along its axis
---------------	---

#### Adjust Image

Shift + Z	Zoom out
-----------	----------

Cmd + R	Reset zoom
---------	------------

Option + →	Rotate image clockwise
------------	------------------------

Shift + R, or Option + ←	Rotate image counterclockwise
--------------------------	-------------------------------

Shift + U	Mirror image vertically
-----------	-------------------------

Cmd + T	Adjust tooth range
---------	--------------------

#### Workspaces

(fn+) F2	3D patient / Dental scan
----------	--------------------------

(fn+) F3	3D panoramic
----------	--------------

(fn+) F4	Tooth <=> Endo
----------	----------------

(fn+) F5	Panoramic
----------	-----------

(fn+) F6	Intraoral
----------	-----------

(fn+) F7	Ceph
----------	------

(fn+) F8	Clinical pictures
----------	-------------------

(fn+) F9	Implant
----------	---------

(fn+) F10	Intraoral camera
-----------	------------------

(fn+) F12	Recent images
-----------	---------------

#### Intraoral Workspace

Shift + Ctrl + F	Change layout to default intraoral workspace template
------------------	---

Shift + Ctrl + C	Change layout to default capturing template
------------------	---

Shift + Ctrl + B	Change layout to vertical bitewing template
------------------	---

### DTX Studio Home (Patient List)

#### Shortcut Description

Esc	Close active patient record
-----	-----------------------------

Shift + P	Export to new patient in DTX Studio Implant
-----------	---

Option + F4	Hide DTX Studio Clinic
-------------	------------------------

### DTX Studio Home (Image Preview)

#### Shortcut Description

Esc	Close preview of the image
-----	----------------------------

Cmd + R	Reset zoom
---------	------------


Shift + R	Rotate 2D image counterclockwise
-----------	----------------------------------

Shift + U	Mirror 2D image vertically
-----------	----------------------------


Shift + Z	Zoom out
-----------	----------

## Mouse Interactions


### DTX Studio Home (Patient List)

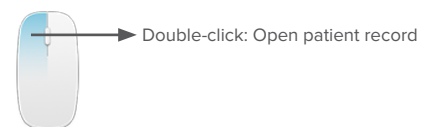

### DTX Studio Home (Image Preview)

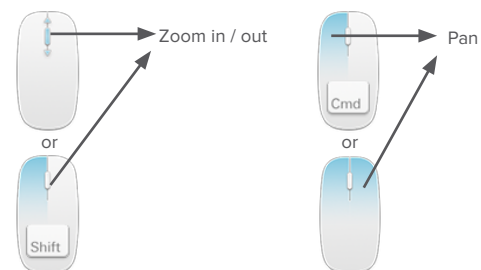

### 2D — DTX Studio Clinic

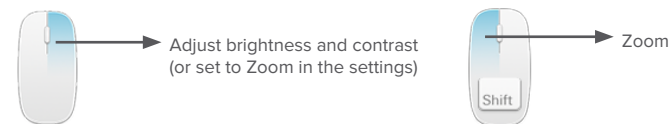

### 3D — DTX Studio Clinic

