

# DTX Studio™ Clinic 3.2 Shortcuts — Windows®



## DTX Studio Home

Esc	F1 Help	F2	F3	F4	F5	F6	F7	F8	F9	F10	F11	F12	Print Screen	Scroll Lock	Pause Break	<ul style="list-style-type: none"> <li>■ Patient List</li> <li>■ Image Preview</li> </ul>					
	1	2	3	4	5	6	7	8	9	0	← Backspace		Insert	Home	Page Up	Num Lock	/	*	-	Zoom out	
Tab	Q	W	E	R	T	Y	U	I	O	P			Del	End	Page Down	7	8	9	+	Zoom in	
Caps Lock	A	S	D	F FMX	G	H	J	K	L Level Window	Enter					4	5	6				
↑ Shift	Z Zoom in	X Take IO X-ray	C Capture	V	B Bite-wings	N	M	<	>	Export images	Shift				1	2	3	Enter			
Ctrl	Alt											Alt Gr	Ctrl			0	.				
													← Prev. image	↑	↓	→ Next image					



## DTX Studio Clinic

Esc	F1	F2	F3	F4	F5	F6	F7	F8	F9	F10	F11	F12	Print Screen	Scroll Lock	Pause Break	<ul style="list-style-type: none"> <li>■ 2D</li> <li>■ 3D</li> </ul>					
	1	2	3	4	5	6	7	8	9	0	← Backspace		Insert	Home First slide	Page Up 10 up	Num Lock	/	*	-	Zoom out	
Tab	Q Close viewer	W	E Crisp filter	R	T Text	Y	U	I	O	P			Del	End Last slide	Page Down 10 down	7	8	9	+	Zoom in	
Caps Lock	A Edit axis	S Spot-light	D	F FMX Point Focus	G	H	J	K	L Level Window	Enter					4	5	6				
↑ Shift	Z Zoom in	X Take IO X-ray	C Capture	V	B Bite-wings	N	M Measure	<	>	Import images	Shift				1	2	3	Enter			
Ctrl	Alt	SmartFocus SmartFocus										Alt Gr	Ctrl			0	.				
													← Prev. image	↑ 1 slide up	↓ 1 slide down	→ Next image					

Adjust brightness and contrast

Rotate 3D view

# DTX Studio™ Clinic 3.2 Shortcuts — Windows®



## Extended Shortcuts

### DTX Studio Clinic

Shortcut	Description
----------	-------------

#### General Shortcuts

Esc	– Restore down maximized window – Finish Spotlight action – Close dialog or wizard
-----	--

Alt + F4	Exit application
----------	------------------

Ctrl + Y	Redo
----------	------

Ctrl + Z	Undo
----------	------

Ctrl + S	Save diagnosis
----------	----------------

#### Capture

Enter	Capture with intraoral camera (when the livestream is active)
-------	---

#### Viewer Options

Ctrl + I	Show / Hide viewer information
----------	--------------------------------

Ctrl + M	Maximize / Restore down
----------	-------------------------

Ctrl + P	Print viewer images and content
----------	---------------------------------

#### Object Interactions

Backspace, or Del	Delete
-------------------	--------

↑ ↓ → ←	Translate the selected implant in the plane of the reslice
---------	--

Ctrl + ↑ ↓ → ←	Translate the selected implant along its axis
----------------	---

#### Adjust Image

Shift + Z	Zoom out
-----------	----------

Ctrl + R	Reset zoom
----------	------------

Alt + →	Rotate image clockwise
---------	------------------------

Shift + R, or Alt + ←	Rotate image counterclockwise
-----------------------	-------------------------------

Shift + U	Mirror image vertically
-----------	-------------------------

Ctrl + T	Adjust tooth range
----------	--------------------

#### Workspaces

F2	3D patient / Dental scan
----	--------------------------

F3	3D panoramic
----	--------------

F4	Tooth <=> Endo
----	----------------

F5	Panoramic
----	-----------

F6	Intraoral
----	-----------

F7	Ceph
----	------

F8	Clinical pictures
----	-------------------

F9	Implant
----	---------

F10	Intraoral camera
-----	------------------

F12	Recent images
-----	---------------

#### Intraoral Workspace

Shift + Ctrl + F	Change layout to default intraoral workspace template
------------------	---

Shift + Ctrl + C	Change layout to default capturing template
------------------	---

Shift + Ctrl + B	Change layout to vertical bitewing template
------------------	---

### DTX Studio Home (Patient List)

#### Shortcut Description

Esc	Close active patient record
-----	-----------------------------

Shift + P	Export to new patient record in DTX Studio Implant
-----------	--

Alt + F4	Hide DTX Studio Clinic
----------	------------------------

### DTX Studio Home (Image Preview)

#### Shortcut Description

Esc	Close preview of the image
-----	----------------------------

Ctrl + R	Reset zoom
----------	------------


Shift + R	Rotate 2D image counterclockwise
-----------	----------------------------------

Shift + U	Mirror 2D image vertically
-----------	----------------------------


Shift + Z	Zoom out
-----------	----------

## Mouse Interactions


### DTX Studio Home

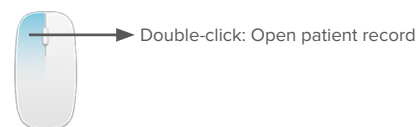

### Image Preview in DTX Studio Home

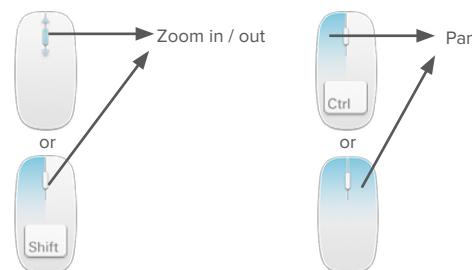

### 2D — DTX Studio Clinic

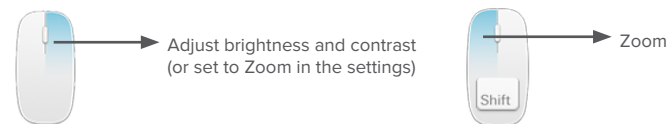

### 3D — DTX Studio Clinic

