

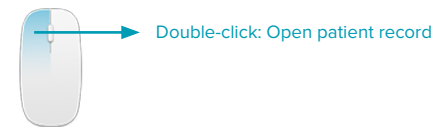
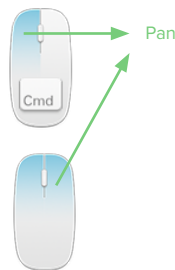


# Shortcuts in DTX Studio™ Clinic 2.3 for Mac

## Patient Module

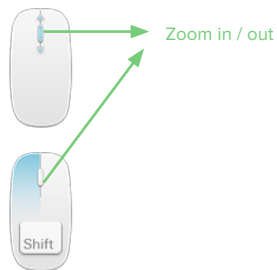

■ Patient Module  
■ Image Preview

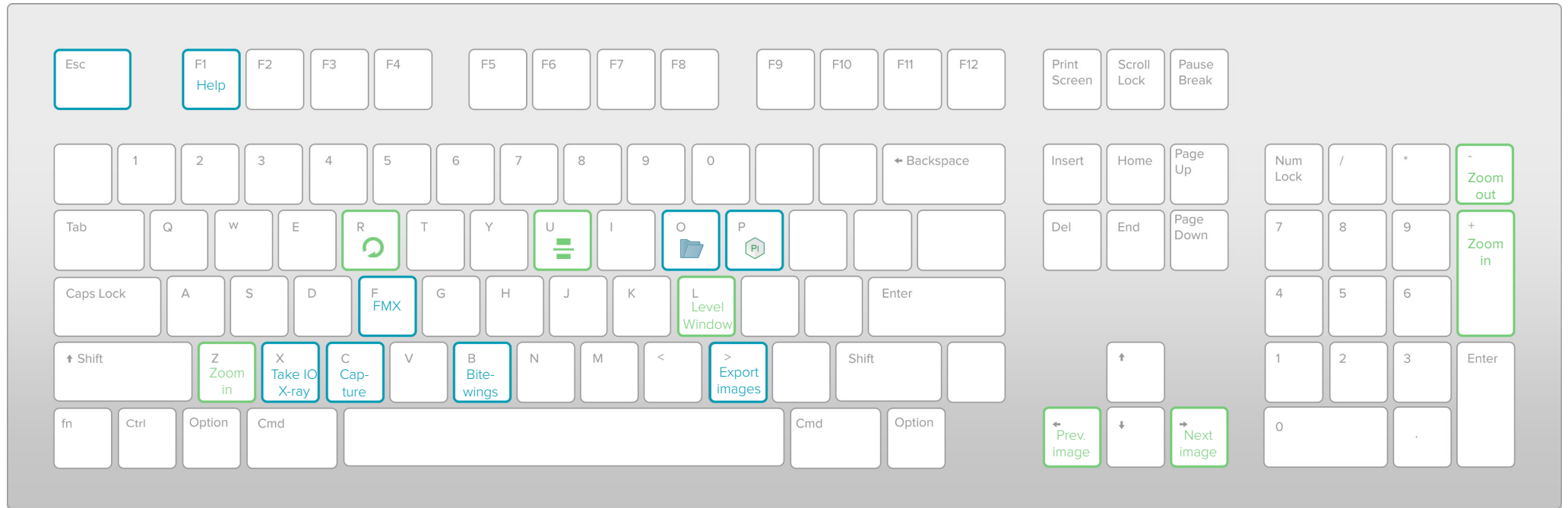

# Shortcuts in DTX Studio™ Clinic 2.3 for Mac






## Open Patient Record



Switch workspaces:

(fn +) F2: 3D patient / Dental scan	(fn +) F7: Ceph
(fn +) F3: 3D panoramic	(fn +) F8: Clinical pictures
(fn +) F4: Tooth	(fn +) F9: Implant
(fn +) F5: Panoramic	(fn +) F10: Intraoral camera
(fn +) F6: Intraoral	(fn +) F12: Recent images

■ 2D  
■ 3D

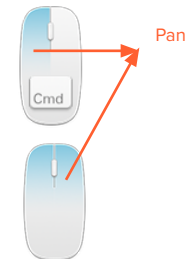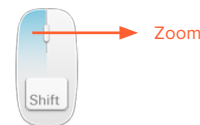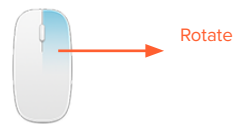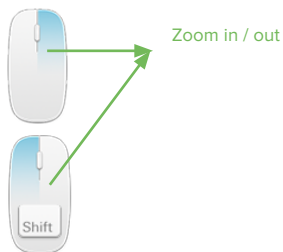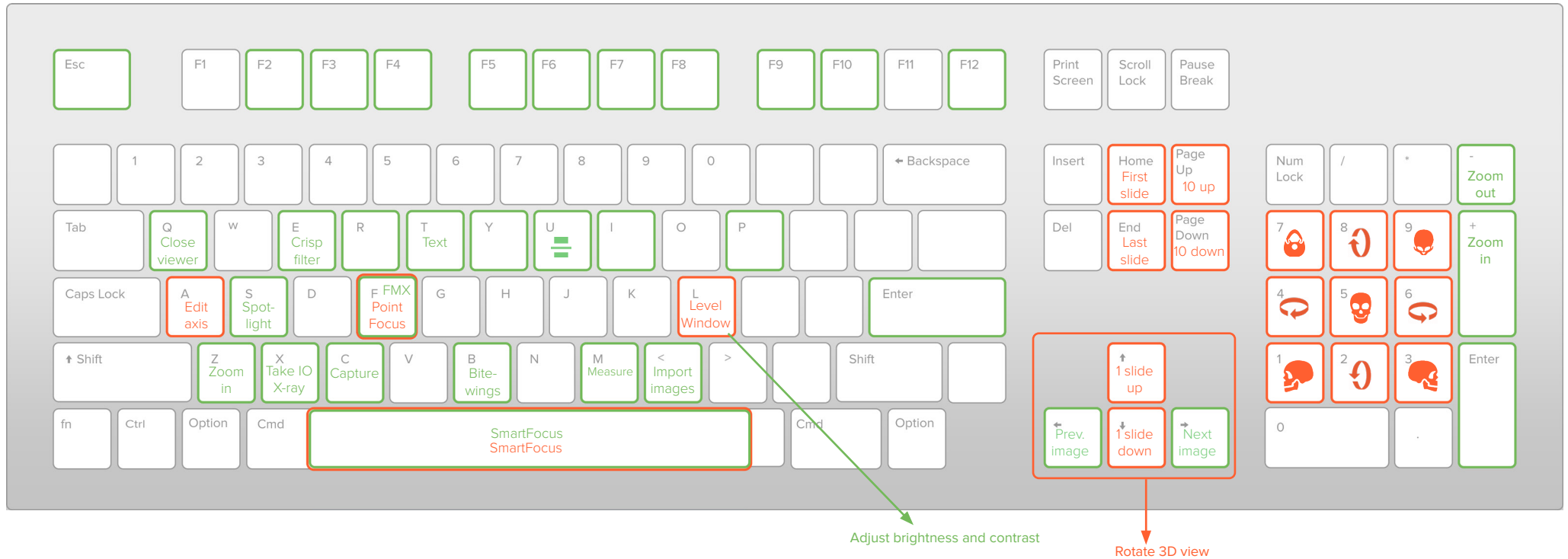


# Shortcuts in DTX Studio™ Clinic 2.3 for Mac

## Extended Shortcuts


### Patient Module

Shortcut	Description
Esc	Close active patient
Shift + P	Export to new patient in DTX Studio™ Implant

### Image Preview in Patient Module

Shortcut	Description
Esc	Close preview of the image
Cmd + R	Reset zoom
Shift + R	Rotate 2D image counterclockwise
Shift + U	Mirror 2D image vertically 
Shift + Z	Zoom out

### Open Patient Record

Shortcut	Description
Esc	– Restore down maximized window – Finish Spotlight action – Close dialog or wizard
Enter	Capture with intraoral camera (when the livestream is active)
Alt + F4	Alt + F4: Exit application
Cmd + I	Show / Hide viewer information
Cmd + M	Maximize / Restore down
Cmd + P	Print viewer images and content
Cmd + Q	Quit application
Cmd + R	Reset zoom
Shift + R	Rotate image counterclockwise
Cmd + S	Save diagnosis
Cmd + T	Adjust tooth range
Shift + U	Mirror image vertically 
Shift + Z	Zoom out
↑ ↓ → ←	Translate the selected implant in the plane of the reslice
Cmd + ↑ ↓ → ←	Translate the selected implant along its axis